



TELESMART

Cleaning of the device

Dear customer,

from time to time our service department receive TELESMART-units damaged by liquids inside the device.

The necessary service is time consuming and expensive: In favourable cases the main board can be dismantled, dried, remounted and tested. In many other cases the mainboard has to be replaced by a new one. Costs can range from ca. € 250,- to more than € 1.000,- .

With this background we would like to draw your attention to the „cleaning and disinfection“ pages of our instructions for use:

Please make sure that during cleaning and disinfection no liquids can enter into the TELESMART unit.

Use an humid but not wet cloth for this purpose and never immerse your TELESMART into water, disinfectant or other liquids.

Please pass this information to your colleagues!

Your Medset Team

This instruction concerns:

article no. AAG 170031 TELESMART – H Recorder

 AAG 170032 TELESMART – P Recorder